
# Cadwell® Sleepmate

Instructions for Use (IFU)

**Pediatric Thermocouple**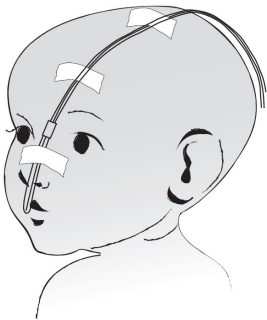

<b>Contents</b>	<b>Page</b>
English .....	4
Dansk .....	6
Deutsch .....	8
Español .....	10
Suomi .....	12
Français .....	14
Italiano .....	16
Nederlands .....	18
Norsk .....	20
Português .....	22
Svenska .....	24

## Intended use

Cadwell Sleepmate™ Reusable Airflow Sensor is intended to detect patient nasal and/or oral breathing during sleep research and clinical studies of sleep.

## Instruction for use

The sensor should be installed so that the probe is exposed to the maximum available airflow. The probes should not touch the skin as contact may attenuate the signal amplitude. To attach to the infant simply tape to the bridge of the nose, such that the sensor hangs in front of the mouth. Run the wire up and over the infant and secure with tape to avoid any entanglement.

Bend and shape sensor prong to fit comfortably on bridge of nose. Use Neonatal approved tape to secure in place.

Important: Be sure tip of sensor is positioned in front of mouth

Connect the sensor to the appropriate channel input of your recorder.

Ensure the sensor is securely attached to the patient as described and you have clear, reliable signals prior to lights out.

## Cleaning

This sensor was designed to be reusable and can be cleaned by rubbing with any cleansing agent that is not corrosive toward plastic. **The entire assembly must be completely dry before using again to insure patient safety.** For sterilization you should implement a standard procedure for gas sterilization. Do not autoclave this product.

## Technical Specifications

Description	Properties
Lead wire	Two conductor white lead wire, approx. 200 cm (80") long
Sensor	Flexible, rubber coated probes for oral (one probe) and nasal (two probes) airflow.
Connectors	Two 1.5 mm (0.1") safety pins
Filter Settings	Low Hz = 0.1 Hz; High Hz = 15 Hz; Gain: Output range is +/- 100 Microvolts. Sensitivity is dependent on variables such as probe placement, patient airflow, ambient room temperature, CPAP pressure, etc.

### Caution Statement:

US Federal Law restricts this device to sale by or on the order of a physician. This product is for diagnostic purposes only and is not intended to be used as an apnea monitor or in a life supporting or sustaining capacity

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©2021 Cadwell Industries, Inc. All rights reserved. PN 302993-639-4 Rev. 00
**Cadwell Industries Inc.**
1 (800) 245-3001 U.S.A. toll-free | +1 (509) 735-6481
This document contains trademarks that belong to Cadwell Industries, Inc.